










# AASRAA BATHEASE ASSEMBLY INSTRUCTIONS

S NO	INSTRUCTION	IMAGE
1	Take The Seat & Keep It Bottom Up	
2	Take The Part Labelled 1 and keep it over the slot diagonally with the Square{ressed Side Facing Upwards	
3	Take The Part Labelled 2 and keep it on the Part 1, Rounded Side On The Top	
4	There are two different sized bolts with the shower chair, the smaller one is M6x33 and the Larger one is M6x54 along with an allen key to tighten the bolts	
4	Take The Smaller Bolt M6x33 And Pass it through The Centre and tighten using the Allen Key	

5	<p>Take The Parts 3 &amp; 4 and align with the X Formation of Pipes 1 &amp; 2, in such a way that the pipes other face face above the plastic seat in the opposite direction (as shown).</p>	
6	<p>Use Larger Bolts M6x54 To Attach Them In Place using the Allen Key</p>	
6	<p>Attach The Backrest To The Other End Of the Pipes 3 &amp; 4 Using the Shorter M6x33 Bolts</p>	
7	<p>Attach The Side Handles To the Chair As Shown</p>	
8	<p>Attach The Walking Stick Holder To The Side Of the Handle If Needed</p>	