आपका सहारा , हमारा *aasraa*
Your support, our assurance

WELCOME TO THE **AASRAA FAMILY**

Thank you for allowing Aasraa to be with you as you move with confidence and freedom.

USER MANUAL

WWW.AASRAA.CO.IN

Warranty

Your purchase from Aasraa guarantees you 1 year warranty that covers any damage caused to the wheelchair under normal usage.*

ACTIVATE WARRANTY

SCAN THE QR CODE

*T&C apply. Available on website.

Your Feedback is Truly Valued

Your trust means a lot to us. When you share how your Aasraa product helped you or made your day easier, it inspires our entire team. We genuinely appreciate every kind word, every photo, and every story you choose to share.

If you are happy with your purchase, you can leave us a review along with text, images, or a short video on the marketplace where you ordered the product.

Upload a screenshot using the QR and enjoy a small cashback as a warm thank you from us.*

Customer Support

We always aim to give you the best experience.

Still, we understand that sometimes things may not go as expected.

If anything feels off or you are facing an issue, our friendly support team is here to help and will do everything possible to bring a smile back to you.

Email: **contact@peaar.co.in**

Call / WhatsApp: **+91 828 747 6144**

*Please share the correct UPI Id and always double check it. Our team might call you to confirm the details. Please allow 24-48 hours for the process to complete after form submission.

Usage & Care Instructions

- To open the wheelchair push down the edges of the seat gently.
- To fold the wheelchair, pull up the seat.
- When sitting or standing up from the wheelchair, always ensure the handbrakes are engaged to restrict movement.
- Keep your arms and feet inside the wheelchair.
- Always use the seat belt when sitting in the wheelchair.
- Always ensure an attendant is supporting the wheelchair on incline and decline areas.
- DO NOT stand on the wheelchair.
- DO NOT use the footrest as a platform. When getting in or out of the wheelchair, make sure the footrest is raised.
- Avoid rough and patchy roads and waterlogged areas.
- Don't pick up or carry the chair.
- Use a soft, damp cloth for cleaning, and avoid harsh chemicals.
- Check and tighten knobs regularly.
- Store in a dry place away from water.
- Replace worn parts when needed.

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